

7 SAUFLON

FORMULATED FOR USE WITH:

- All Soft Contact Lenses
- Especially suitable for Continuous wear contact lenses.



- 1 REFRESHER
- 2 COMFORT DROP
- 3 EMERGENCY DISINFECTION
- 4 LENS STORAGE
- 5 CLEANING
- 6 RINSING
- 7 RE-WETTING

7 USES

Refresher

There are some days when lenses are less comfortable on the eye, either because of the environment, working conditions, the weather, or even the way the patient is feeling on that particular day. On such occasions Sauflon 7 can be used to freshen up the lens either by applying a few drops directly in to the eye or by rinsing the lenses in a cupped palm. This simple exercise will freshen the lens with a retentive aqueous film.

Comfort Drop

Placing a few drops of Sauflon 7 into the bowl of all soft contact lenses assists initial comfort. This will be particularly noticeable with some Daily Disposable contact lenses. Long term comfort is greatly improved for both extended wear and daily disposable contact lens wearers, particularly if the lenses are removed, rubbed and rinsed and then reinserted.

Dehydration of lenses towards the end of the day leads to the lenses becoming tight and uncomfortable on the eye. Rehydration of the lenses with Sauflon 7 can considerably extend wearing time.

Emergency Disinfecting

There will be times, particularly with extended wear contact lenses, where the patient will want a rest from wearing. Sauflon 7 can be used for overnight disinfection on these occasions.

Lens storage

Sauflon 7 can also be used as a contact lens storage solution. However if storing for longer than seven days the lenses should be rubbed and rinsed with the solution and then disinfected overnight before wearing.

Cleaning

Sauflon 7 contains a gentle ocular grade surfactant which has been proven to be an exceptionally efficient cleaning agent.

Rinsing

Packed in a convenient travel size, Sauflon 7 is the perfect solution to take on trips or carry on a daily basis. Ideal for rinsing debris from lenses or the surface of the eye at anytime.

Re-Wetting

Sauflon 7 is the perfect solution to use for re-wetting the lens at anytime throughout the day. The product temporarily clings to the corneal & lens surfaces providing lubrication for the lens and a cushioning, comforting effect on the eye.



7 SAUFLON

