

SAUFLON

COMFORT DROPS

for contact lens wearers



- 💧 Improves lens comfort
- 💧 Extends wearing time
- 💧 Lubricates the lens
- 💧 Use with contact lenses in the eye
- 💧 Suitable for all types of contact lenses
- 💧 Excellent value

 **SAUFLON** Only for opticians
The company you can trust

Sauflon Pharmaceuticals Ltd
49-53 York Street, Twickenham TW1 3LP England
Telephone 020 8322 4200 Fax 020 8891 3001



LENS INSERTION

All soft Lenses - Placing a few drops in bowl of lens will assist initial comfort. Particularly noticeable with One Day Acuvue.

Daily Disposable - Can be important as no solution available to assist insertion.

Rigid Lenses - Covering both surfaces before insertion will lead to marked improvement in comfort.



IMPROVED WEARING COMFORT

Comfort greatly improved with all soft lenses, particularly if lens is removed, rubbed and rinsed and then reinserted. Quite dramatic improvement can be experienced with toric lenses and ionic lenses such as *Surevue*, *Acuvue* and *Focus*.



EXTENDED WEARING TIME

Dehydration towards end of day leads to lenses becoming tight and uncomfortable on the eyes. May be very noticeable with ionic lenses and toric lenses. Use of **Sauflon Comfort Drops** can be particularly important for One Day Disposable wearers who may wear their lenses all day in the office and plan to go out in the evening, by which time their lenses are tight and uncomfortable.



IMPROVED VISUAL ACUITY

Visual acuity can fall with all soft lenses due to parameter changes associated with dehydration.

As much as a complete line can be recovered with the use of **Sauflon Comfort Drops** - even more with torics.



EXCELLENT VALUE

A very high quality product deliberately offered at a very low patient price in order to encourage regular usage.