



# CyClean™

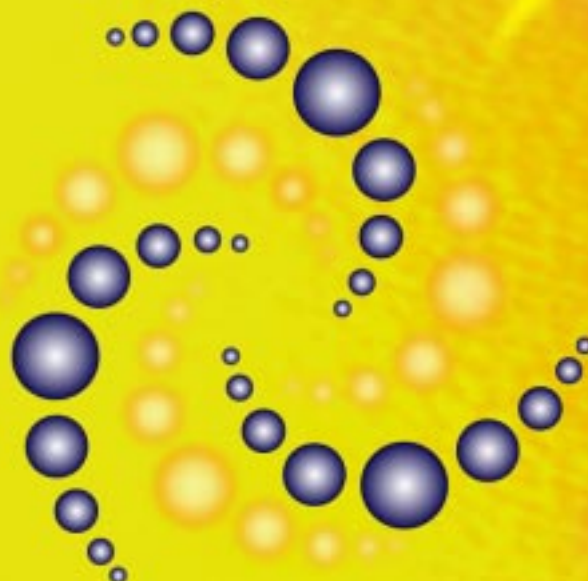

A completely new, unique and  
revolutionary contact lens  
solution

**A brand new genuine no rub,  
no rinse multipurpose solution  
formulated by Sauflon, to  
make caring for lenses safer  
and easier than ever.**

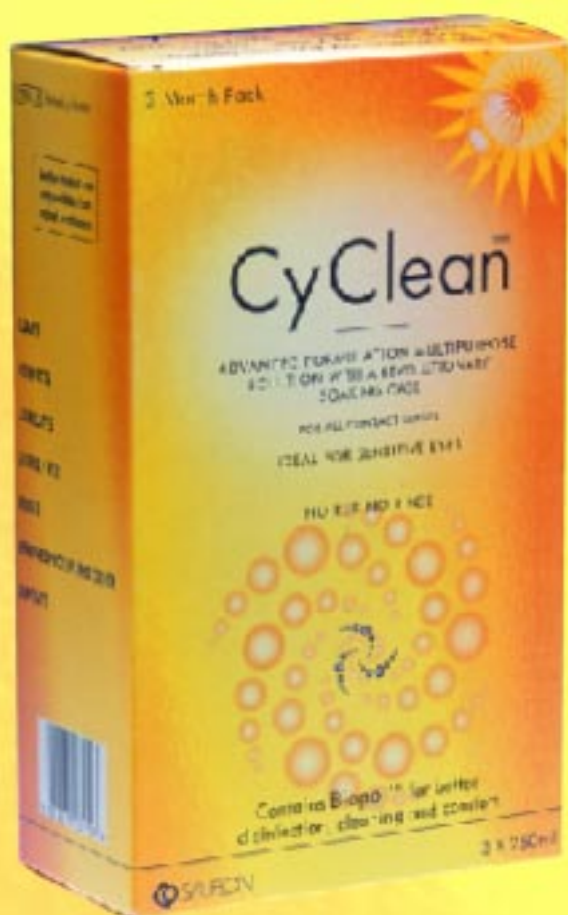

**New advanced formulation  
contains **Biopol™** for  
enhanced disinfection,  
cleaning and comfort.**

# The Biopol™ Effect

REVOLUTIONARY  
SOAKING CASE

## Biopol™

Twist the cap...




Biopol™ is a unique synergistic combination of active ingredients, developed by Sauflon to enhance the cleaning, disinfecting, comfort and anti acanthamoebal activity.

...clean the lens!

**Combine to create the most advanced lens care product available today.**

Twisting the cap of the **CyClean Case** creates a vortex action which cleans the whole lens and not just the central portion.

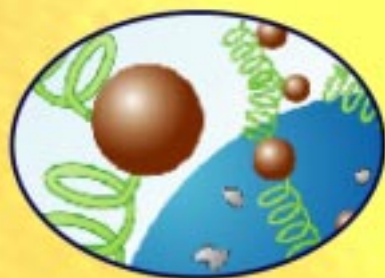

Soiled Lens



Breakdown & Dispersion




Cleaned Lens

The above diagrams illustrate how the **Biopol™** breaks down protein/calcium bridges and disperses lipids, leaving a clean lens which extends wearing time.

# Better Cleaning

**Graph 1 Protein Removal**

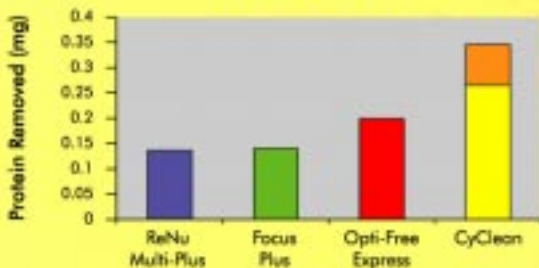


Graph 1 illustrates that CyClean removes significantly more protein than its nearest competitors.

**Graph 2 Lipid Removal**



CyClean is extremely effective in removing lipids from the lens, as shown by the test results illustrated in Graph 2.


Source: Independent Studies (data on file).

 **The performance of CyClean is increased by up to a further 30% when using the twisting action of the soaking case.**

# Better Disinfection

**Graph 3**

■ Pseudomonas aeruginosa  
■ Serratia marcescens  
■ Staphylococcus aureus

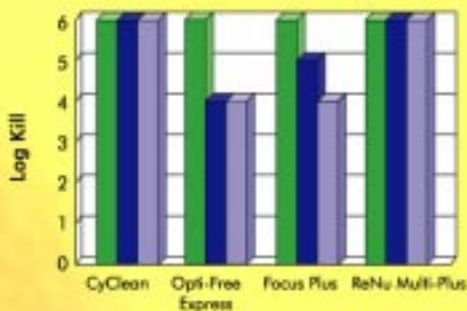


Following an inoculation of 1,000,000 (6 log) of each organism, Graph 3 shows the level of kill achieved for each organism in the manufacturers' recommended soak time. (6 log is complete kill).

Source: Independent Laboratory studies using EN ISO 14729: 2001 Standard Disinfection Efficacy Stand alone Method.

# Kills Acanthamoeba

**Graph 4 CyClean against Acanthamoeba polyphaga (Ros) cysts (MgC12)**

—■— CyClean    —●— Control

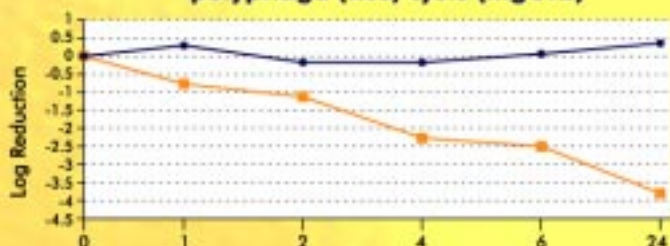

Independent Studies carried out at Leicester University shows that CyClean is effective against the more resistant cyst form of Acanthamoeba.

# Better Comfort

The synergistic combination of active ingredients within **Biopol™** enhances lens comfort by lubricating and wetting the lens surface.

# Benefits



This unique solution provides Practitioners with the opportunity to prescribe the latest innovation in contact lens care.



The CyClean Case, twist the cap - clean the lens - means less handling, leading to better hygiene and less risk of damage to the lens.



The CyClean Case ensures that the entire lens surface is cleaned everytime, not just the central portion. The CyClean Case therefore eliminates the need to rub and rinse ensuring better patient compliance.



**Biopol™** enhances disinfection efficacy leading to better kill times against ocular pathogens including acanthamoeba.



This unique formulation, containing **Biopol™**, ensures outstandingly comfortable lens wear, increasing patient satisfaction leading to a lower level of drop outs.



Optician Only - CyClean is only available to Optical Practitioners



# SAUFLON

**Only for opticians**



**Tel.: 020 8322 4200**  
**Fax: 020 8891 3001**  
**E-mail: [info@Sauflon.co.uk](mailto:info@Sauflon.co.uk)**